



## Operative Technique

5.6mm, 7.4mm Headless and Headed  
Cannulated Compression Screws.

Sterile, Disposable Instrumentation.



## CONTENTS

---

|                                |    |
|--------------------------------|----|
| 1. Indications .....           | 3  |
| 2. Design Features.....        | 4  |
| Technical Specifications ..... | 5  |
| Ortholuculent Implants.....    | 6  |
| 3. Applications .....          | 7  |
| 4. Operative Technique .....   | 8  |
| 5. Catalog Information .....   | 13 |
| Instruments Layout.....        | 17 |

### Contributing Surgeons:

**Ettore Vulcano, M.D.**

Miami Beach, FL

**Amiethab A Aiyer, M.D.**

Baltimore, MD

This document offers technical guidance pertaining to the Creed Ortholuculent Implants. As with any medical device, surgeons should rely on their training, making any necessary adjustments based on the needs of the patient.

## Indications

### Indications for Use and Intended Use:

CREED™ Ortholucent Implants are intended to maintain alignment and fixation of bone fractures, comminuted fractures in the presence of appropriate additional immobilization such as rigid fixation implants, cast or brace, non-unions, osteotomies, arthrodesis or bone grafts in the hand, foot, and ankle including distal tibia and fibula. These implants are not intended for spinal use.

## Contraindications

Severe muscular, neurological or vascular deficiency in the extremity concerned.

Bone destruction or poor bone quality, likely to impair implant stability.

Surgical procedures other than those listed in the Indications section.

Known or suspected allergy to any of the device components.

Use of this implant together with implants of another origin not recommended by GLW, Inc.

## Precautions

If either the implant or the package appears damaged the implant should not be used.

Meticulous preparation of the implant site and selection of the proper size implant increase the potential for a successful outcome.

# Design Features

Created in conjunction with foot and ankle specialists, the Creed Ortholucent Implants are designed to deliver maximum compression with minimum torque to address a large variety of osteotomies, fusions and fractures.

## Reverse-Cutting Design


Expedites screw extraction

## Forward-Cutting Design

Accelerates screwhead countersinking

## Ti Core with Anti-Rotation Hex

Provides a strong and stable construct

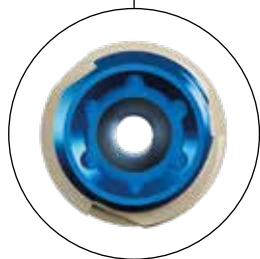


**Torx Interface**  
Promotes a sturdy driver / screw interface



**Self-Drilling Tip Design**

Streamlines screw insertion

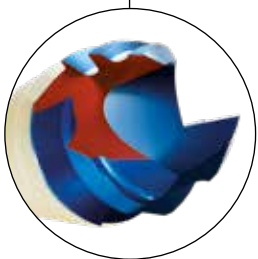

**Self-Tapping Tip Design**
















**Titanium Reverse-Cutting Feature**

Facilitates removal of screw

# Design Features – Technical Specifications

Creed Ortholuculent Implants are available in four diameters and offer a wide range of lengths with 2mm and 5mm increments:

| Ø Size Range | 2.5mm   | 4.3mm   | 5.6mm   | 5.6mm   | 7.4mm   | 7.4mm   |
|--------------|---|---|---|---|---|---|
| Material     | Ti6Al4V / PEEK  |   |   |   |   |   |
| Type         | Headless<br>Compression<br>Screw<br> | Headless<br>Compression<br>Screw<br> | Headless<br>Compression<br>Screw<br> | Headless<br>Compression<br>Screw<br> | Headless<br>Compression<br>Screw<br> | Headless<br>Compression<br>Screw<br> |
| Thread       | 8mm   | 10mm  | 16mm  | 32mm  | 19mm  | 32mm  |
| Length       | Headed: 12-30mm<br>Headless: 14-30mm<br>each 2mm  | From 18-50mm<br>each 2mm<br>From 50-60mm<br>each 5mm  | From 30-50mm<br>each 2mm<br>From 50-80mm<br>each 5mm  | From 42-50mm<br>each 2mm<br>From 50-80mm<br>each 5mm  | From 40-50mm<br>each 2mm<br>From 50-120mm<br>each 5mm   | From *40-50mm<br>each 2mm<br>From 50-120mm<br>each 5mm<br><small>*Headless begin at 46mm</small>                        |
| K-wire       | Ø1.3mm x 150mm  | Ø2.0mm x 150mm  | Ø2.8mm x 230mm  | Ø2.8mm x 230mm  | Ø3.2mm x 230mm  | Ø3.2mm x 230mm  |
| Torx Tip     | T8                                   | T15                                  | T25                                  | T25                                 | T30                                  | T30                                  |


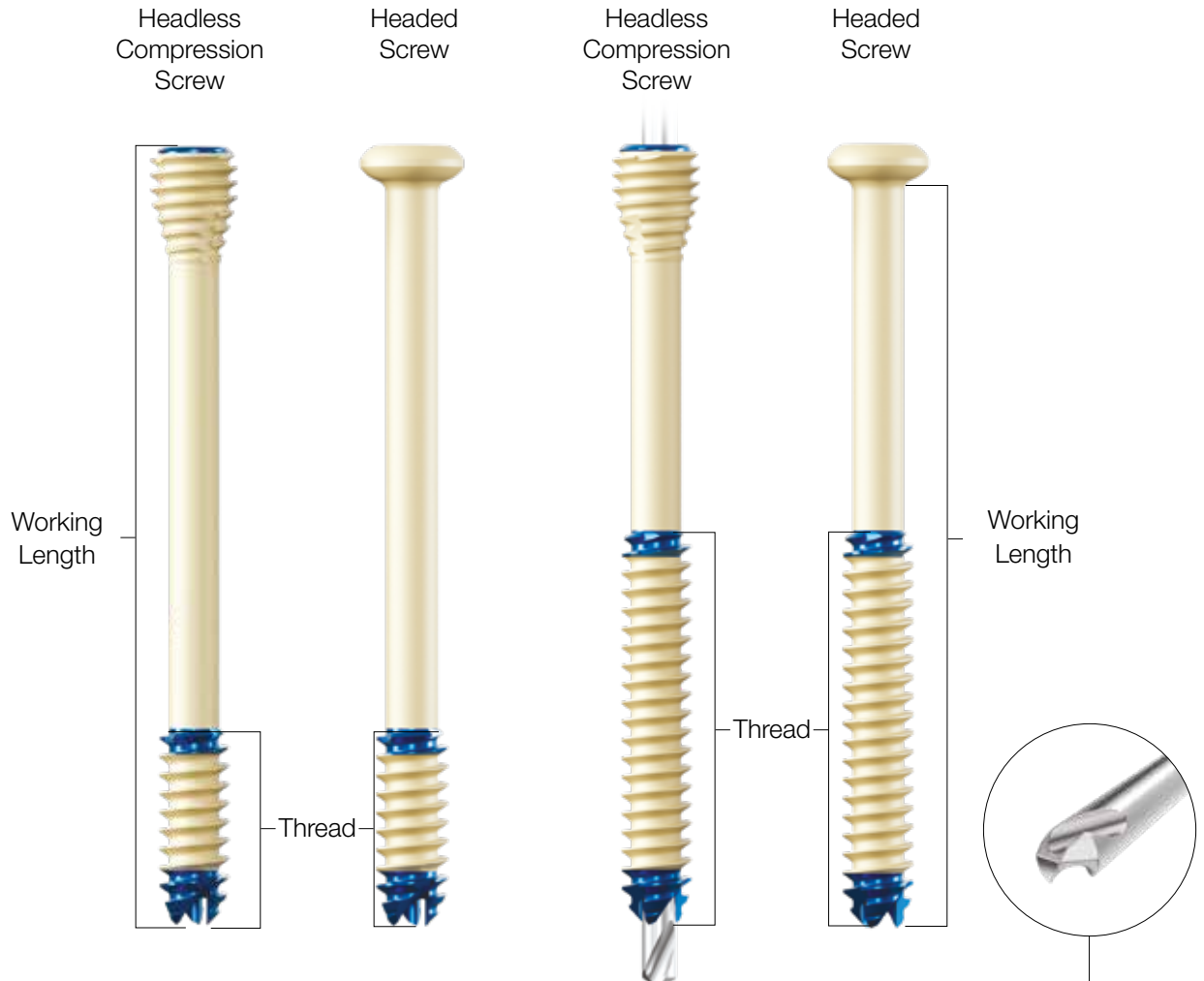


# Design Features – Ortholucent Implants

The radiotranslucent properties of the Creed Ortholucent Implants provide a significant clinical advantage over traditional metal implants by drastically improving the visualization of bones and joint spaces. Surgeons can better assess post-operative healing by “seeing through” the implant and down to the bony structures on radiographs.



The lower titanium load, which is reinforced by a polyetheretherketone (PEEK) overmold, provides clear bone visibility during plain radiography as well as less scatter with advanced imaging techniques. Improved visualization enables the clinician to interpret bone healing with greater confidence leading to faster advancement of recovery protocols.

Due to the radiolucent properties of PEEK, the threads of the Creed Ortholucent Implants are invisible on radiographs. To identify where the screw threads end, the Creed Ø5.6mm and Ø7.4mm screws have a titanium reverse cutting geometry that performs as a marker on X-rays. The Ø4.3mm screws have titanium ring markings located underneath the PEEK reverse cutting geometry while the Ø2.5mm screws have titanium threads and do not require this feature.

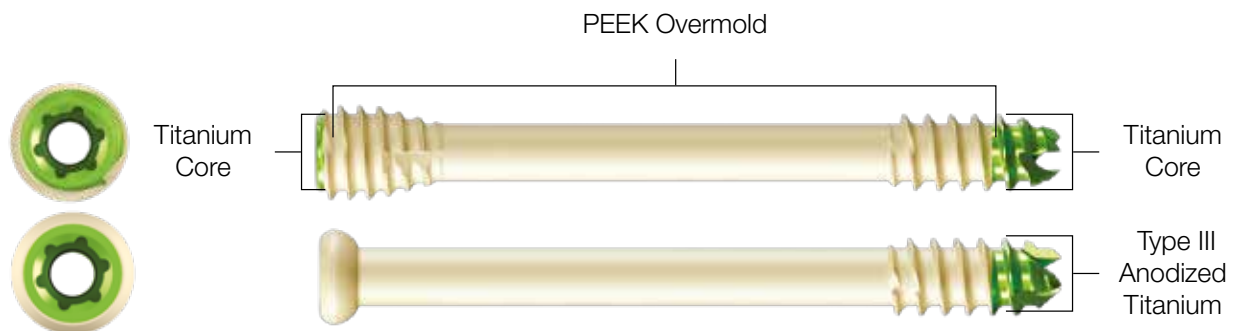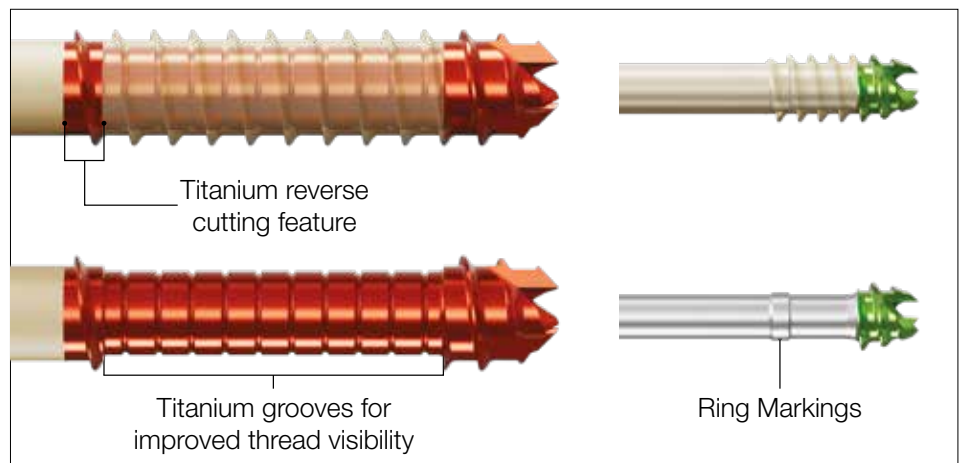


# Applications

| Ø Screw Size                            | ■ Ø2.5mm       | ■ Ø2.5 /<br>■ Ø4.3mm     | ■ Ø4.3mm                 | ■ Ø5.6 /<br>■ Ø7.4mm   |
|---|----------------|--------------------------|--------------------------|------------------------|
| Procedure/Anatomy                       | A.<br>Forefoot | B.<br>Forefoot / Midfoot | C.<br>Midfoot / Hindfoot | D.<br>Hindfoot / Ankle |
| Akin Osteotomy (Bunion)                 | •              |                          |                          |                        |
| Weil Osteotomy (Metatarsal Shortening)  | •              |                          |                          |                        |
| Austin / Chevron Osteotomy (Bunion)     | •              | •                        |                          |                        |
| Scarf Osteotomy (Bunion)                | •              | •                        |                          |                        |
| MTP Fusion                              |                | •                        | •                        |                        |
| Lapidus Procedures (Bunion)             |                | •                        | •                        |                        |
| TMT Fusion                              |                | •                        | •                        |                        |
| Intercuneiform Fusion                   |                | •                        | •                        |                        |
| Talo-Navicular (TN) Fusion              |                | •                        | •                        |                        |
| Calcaneo-Cuboid (CC) Fusion             |                | •                        | •                        |                        |
| Navicular Cuneiform (NC) Fusion         |                | •                        | •                        |                        |
| Lisfranc Injury                         |                | •                        | •                        |                        |
| Tarsal / Metatarsal Fracture            |                | •                        | •                        |                        |
| Syndesmosis Repair                      |                |                          | •                        |                        |
| Ankle Fracture                          |                |                          | •                        |                        |
| Jones Fracture                          |                |                          | •                        | •                      |
| Triple Arthrodesis                      |                |                          | •                        | •                      |
| Calcaneal Osteotomy / Calc Slide (MDCO) |                |                          | •                        | •                      |
| Ankle Arthrodesis                       |                |                          |                          | •                      |
| Subtalar Fusion                         |                |                          |                          | •                      |

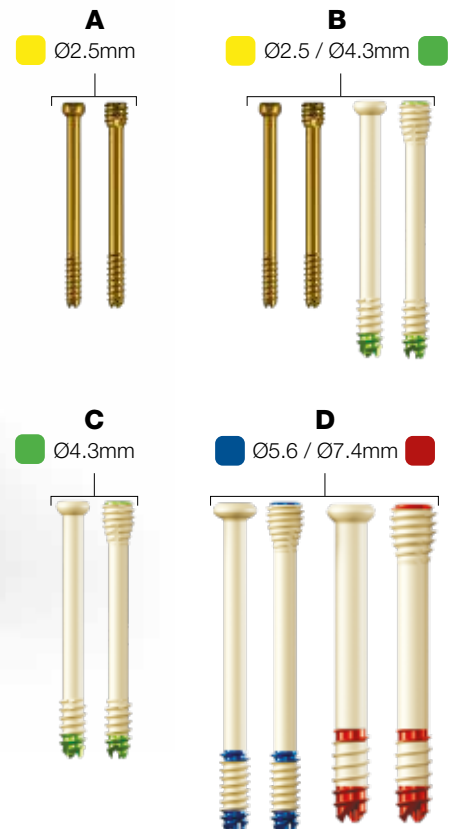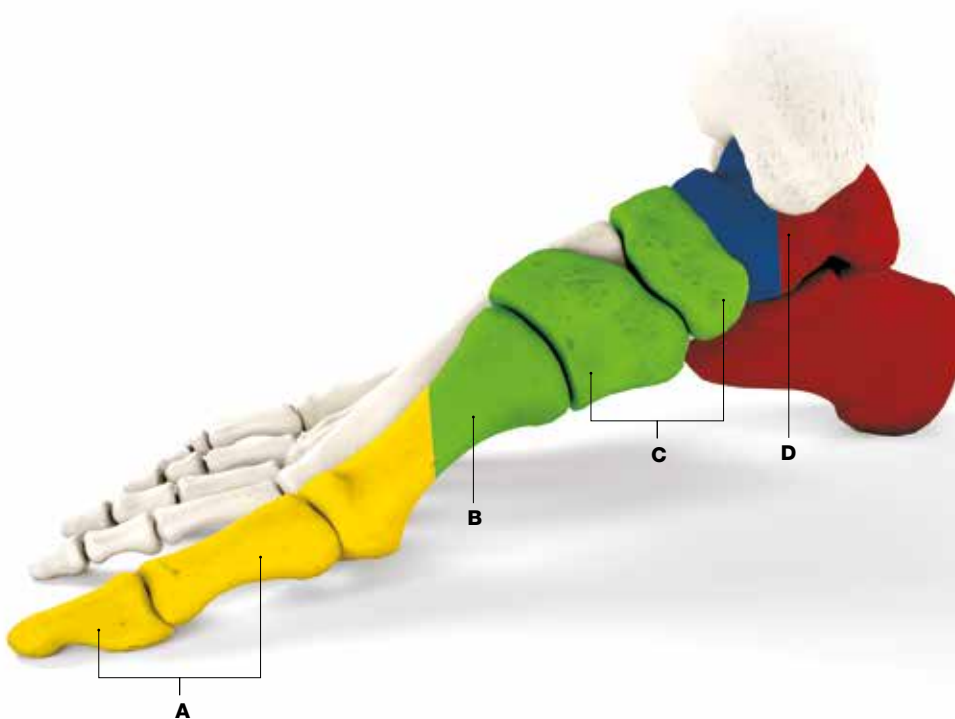




## Applications – Ø5.6mm & Ø7.4mm Screws

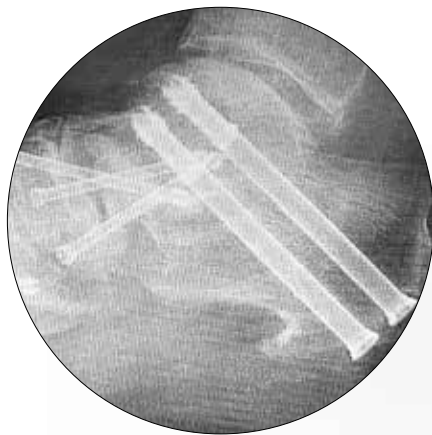


Medial double arthrodesis  
(TN and Subtalar fusion)

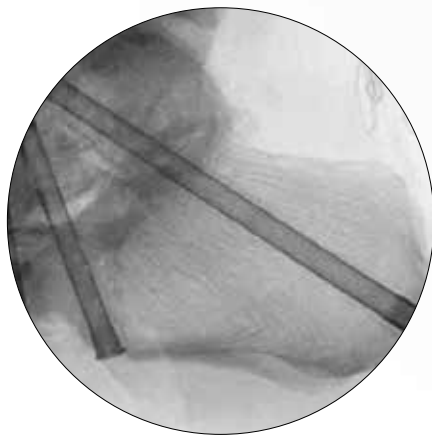

Subtalar fusion





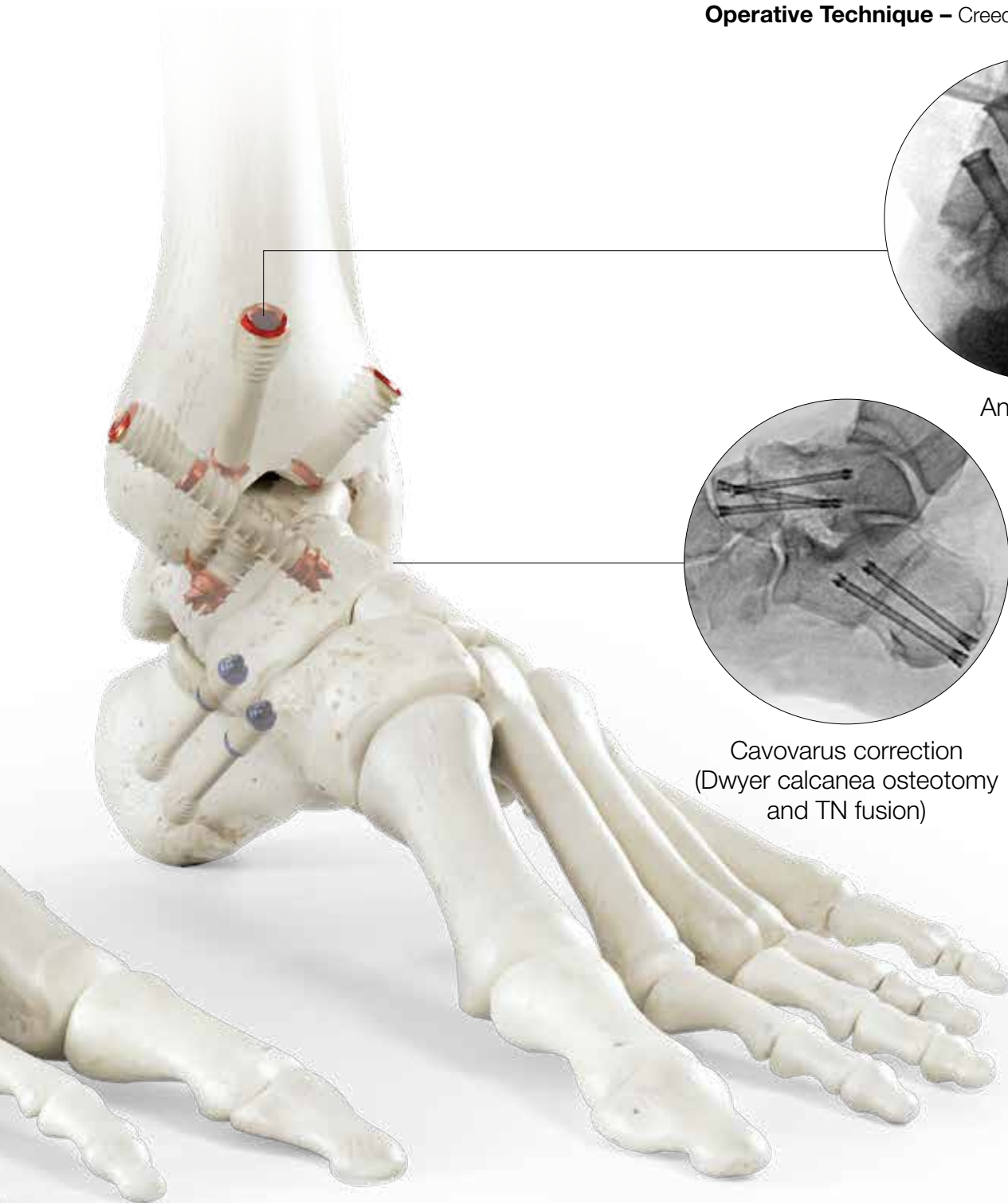

Ankle fusion

Cavovarus correction  
(Dwyer calcanea osteotomy  
and TN fusion)


# Design Features – Surgery-ready Instrument Kits

## CREED Single Use Kits

Creed Ortholucent Implants and instrument kits are sterile packaged, enabling a surgery-ready solution for the hospital and ambulatory surgical centers.

### Large Kit


The large instrument kit contains a T25 and T30 cannulated screwdriver blade for Creed Ø5.6mm and Ø7.4mm screws, respectively, which is used in combination with the Zimmer-Hall-handle. In addition, the large kit includes a Ø6.1mm cannulated drill for Creed Ø7.4mm screws, and a countersink depth gauge with mounted Ø2.8mm and Ø3.2mm K-wires.

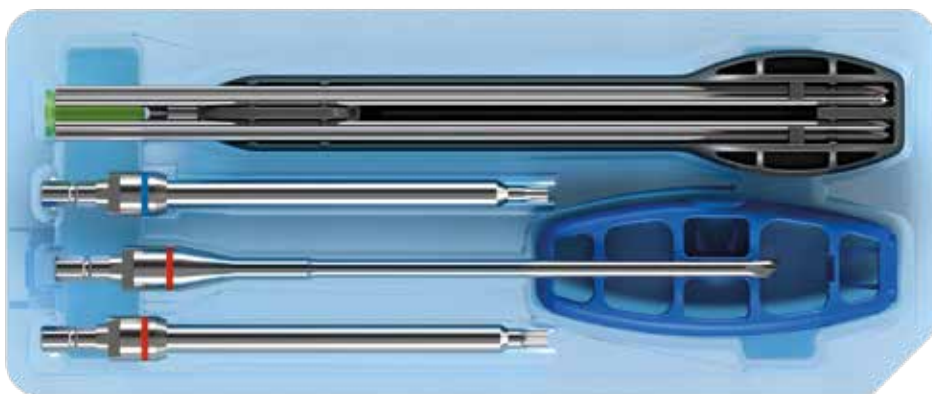


Large Instruments Blister Pack

### Protective cap removal


Each countersink depth gauge is pre-assembled with a protective cap designed to prevent puncturing of the blister. Follow the steps below on how to remove the protective cap correctly.

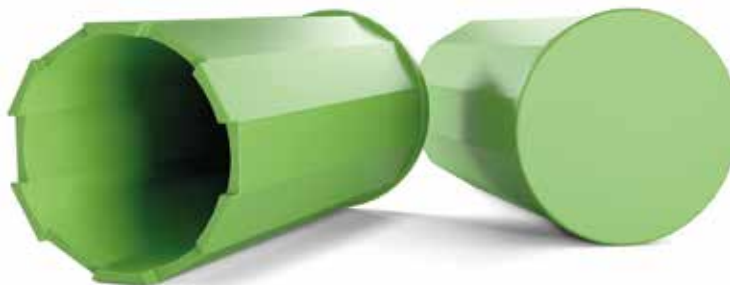

Extract the countersink depth gauge from the blister. With one hand hold the countersink depth gauge and K-wires in place.

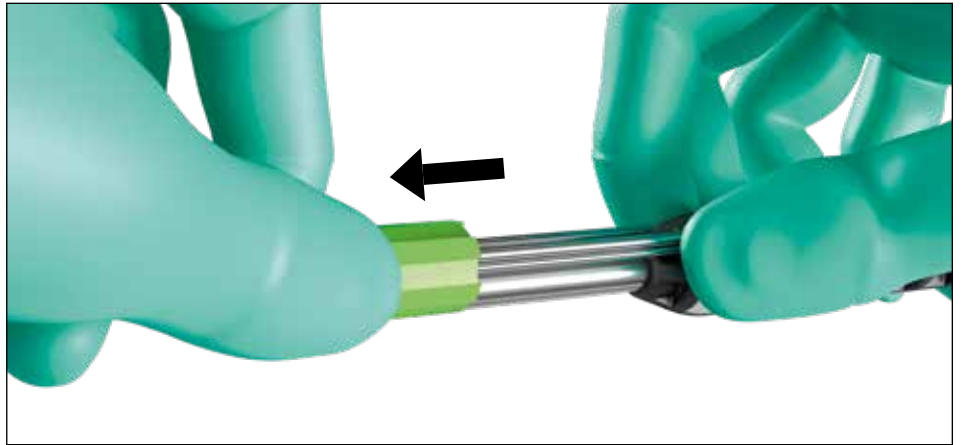

Remove the green cap from the tip of the countersink depth gauge while keeping the wires fixated. Extract the K-wires from the countersink once the cap is fully removed.

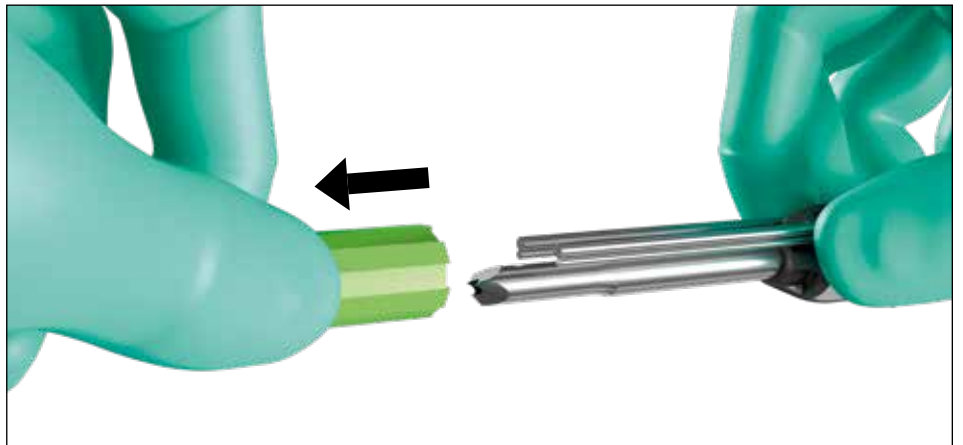

# Operative Technique – Zadek Osteotomy



## Step 1 – Patient Positioning

Patient is positioned in a lateral position with the operative foot at the edge of the bed.

Position the mini c-arm at the contralateral side of the surgical extremity & power equipment at the ipsilateral side of the surgical extremity.

Positioning of OR equipment is a recommendation until a surgeon's preference is identified.



## Step 2 – Mark the Osteotomy


Using fluoroscopy mark the osteotomy site on the lateral aspect of the heel.

A visual drawing of the dorsal closing wedge is recommended. The surgeon should plan to remove 8-10mm of the dorsal closing wedge.

The apex (hinge) of the osteotomy should be 5-10mm proximal to the plantar cortex of the calcaneus.

The osteotomy should be halfway between the insertion of the achilles and the posterior subtalar joint.

in patients with a flatfoot the osteotomy should be more obliquely oriented (from dorsal-posterior to plantar-anterior to prevent further reduction

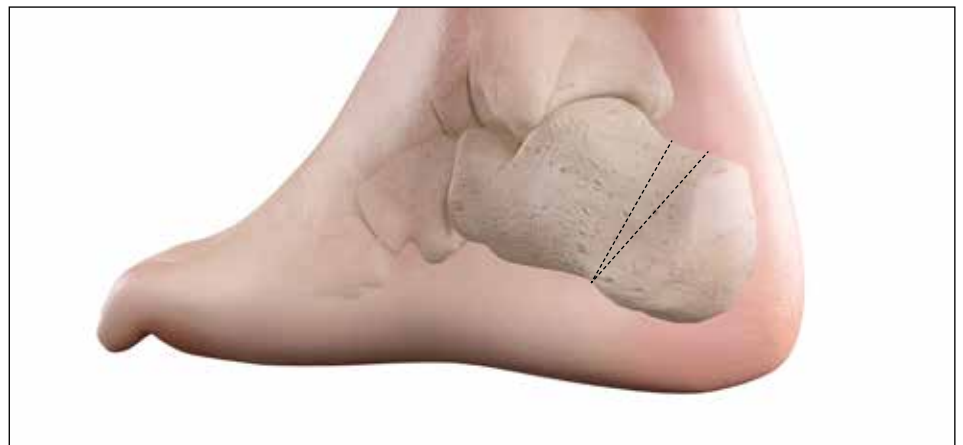

of the calcaneal pitch, whereas patients with cavus alignment would benefit from a more vertically oriented osteotomy to reduce the pitch).


The surgeon may also place two K-wires through the plantar aspect of the calcaneus to create a visual guideline to be seen under fluoroscopy.



### **Step 3 – Incision**

A 3mm incision is made corresponding to the apex of the osteotomy. Bluntly dissect the soft tissues with a hemostat, creating a safe pathway to insert the burr.

Osteotomies performed with a burr, should be powered with a high torque & low rpm motor to avoid complications.

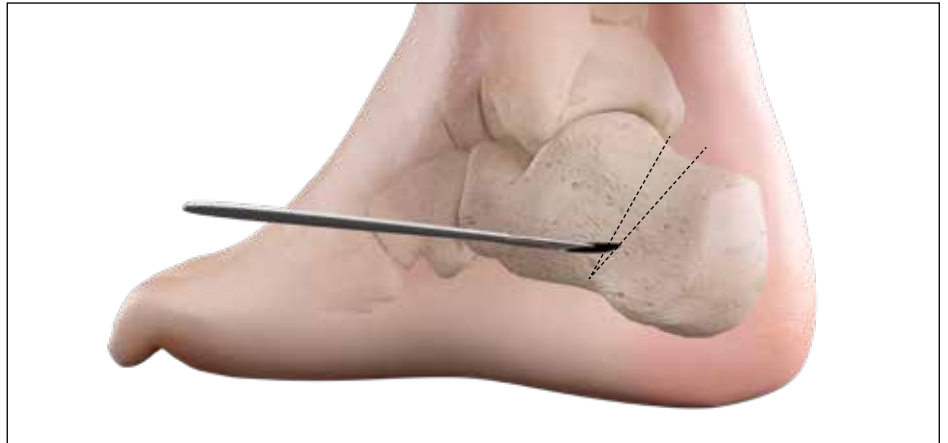

### **Step 4 – Osteotomy**

The osteotomy should be performed in quadrants. Insert the burr and punch through the medial cortex and complete each quadrant before moving on to the next:

Near cancellous bone, far cancellous bone, poke holes in the far cortex, then “connect the dots” along the far cortex to complete the osteotomy.

Once the osteotomy is complete use a 3 or 4 mm wedge burr or a 3x30mm cutting burr to start removing the bone wedge from the dorsal aspect of the osteotomy. Remove 8-10mm.

Once the near cortices are complete, make a series of poke holes along the far cortex to feel the cortex break as the burr passes through.





# Operative Technique – Zadek Osteotomy

## Step 5 – Closing the Osteotomy

To reduce the wedge, maximally dorsiflex the ankle. Insert the guidewire for two Creed Screws for the 7.4mm. Make an incision over the guidewire, measure, drill, countersink and advance selected screw(s).



Make an incision over the guidewire, measure, drill, countersink and advance selected screw(s).





# Operative Technique – Medial Displaced Calcaneal Osteotomy

## Step 1 – Patient Positioning

Patient is positioned supine with the operative foot at the edge of the bed. To internally rotate the operative leg, place a bump under the hip.

Position the mini c-arm at the contralateral side of the surgical extremity & power equipment at the ipsilateral side of the surgical extremity.

Positioning of OR equipment is a recommendation until a surgeon's preference is identified.



## Step 2 – Mark the osteotomy

Under fluoroscopic guidance take a lateral image of the calcaneus. The desired trajectory of the osteotomy can be outlined with the use of a k-wire and a marking pen. The path of the osteotomy should be a safe distance from the posterior facet of the subtalar joint.

A 3mm incision is made at the center of the osteotomy and using a hemostat to create a pathway to introduce the burr.



## Step 3 – Incision

A 3mm incision is made at the center of the osteotomy and using a hemostat to create a pathway to introduce the burr.



## Step 4 – Osteotomy

Osteotomies performed with a burr should always be powered with high torque, low rpm motor, and kept at safe speeds. Be mindful to avoid pulling on the skin by rotating around the fulcrum by supinating and pronating to perform osteotomies.

The calcaneal osteotomy is performed in quadrants, completing each section before moving on to the next.

Introduce the burr into the calcaneus and punch through the medial cortex. The osteotomy is then completed in the following steps:

Near cancellous, far cancellous, poke holes, connect the dots.

Once the near cortices are complete, make a series of poke holes along the far cortex to feel the cortex break as the burr passes through. Bone paste and fluid will begin exiting the incision as evidence the osteotomy has been completed.

## Step 5 – Translation

Once the osteotomy is mobile, introduce a periosteal elevator or instrument of choice into the anterior aspect of the calcaneus. Acting as a lever apply pressure to displace into the desired position.

## Step 6 – Fixation


Introduce two guidewires corresponding to either the 5.6mm or 7.4mm Creed screw to provisionally fixation. Once in the desired position, measure, drill, countersink, and fixate.



# Operative Technique – Medial Displaced Calcaneal Osteotomy


## Step 7 – Screw insertion

Once in the desired position, measure, drill, countersink, and fixate.

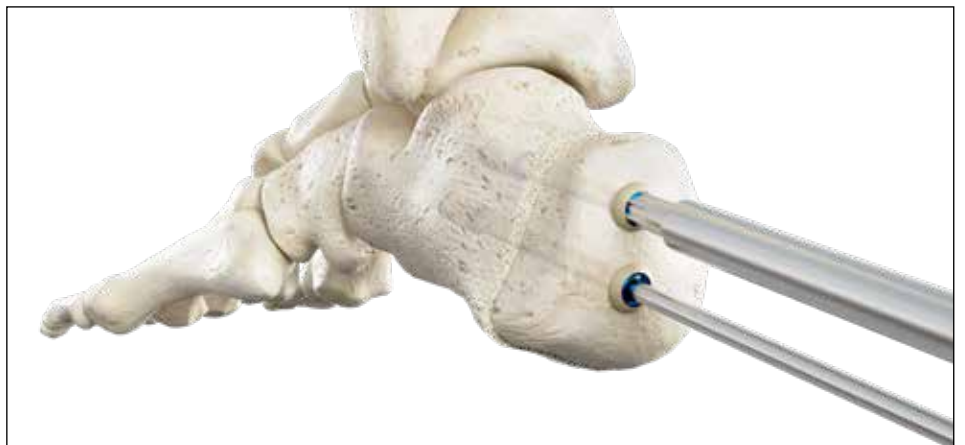

# Catalog Information – Implants

## Ø5.6mm Headless Compression Screw / Headed Screw

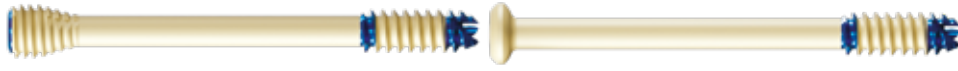

| Headless Ø5.6mm (REF) | Headed Ø5.6mm (REF) | Length (mm) | Thread (mm) |
|-----------------------|---------------------|-------------|-------------|
| F1-1656-030S          | F2-1656-030S        | 30mm        | 16mm        |
| F1-1656-032S          | F2-1656-032S        | 32mm        | 16mm        |
| F1-1656-034S          | F2-1656-034S        | 34mm        | 16mm        |
| F1-1656-036S          | F2-1656-036S        | 36mm        | 16mm        |
| F1-1656-038S          | F2-1656-038S        | 38mm        | 16mm        |
| F1-1656-040S          | F2-1656-040S        | 40mm        | 16mm        |
| F1-1656-042S          | F2-1656-042S        | 42mm        | 16mm        |
| F1-1656-044S          | F2-1656-044S        | 44mm        | 16mm        |
| F1-1656-046S          | F2-1656-046S        | 46mm        | 16mm        |
| F1-1656-048S          | F2-1656-048S        | 48mm        | 16mm        |
| F1-1656-050S          | F2-1656-050S        | 50mm        | 16mm        |
| F1-1656-055S          | F2-1656-055S        | 55mm        | 16mm        |
| F1-1656-060S          | F2-1656-060S        | 60mm        | 16mm        |
| F1-1656-065S          | F2-1656-065S        | 65mm        | 16mm        |
| F1-1656-070S          | F2-1656-070S        | 70mm        | 16mm        |
| F1-1656-075S          | F2-1656-075S        | 75mm        | 16mm        |
| F1-1656-080S          | -                   | 80mm        | 16mm        |
| F1-3256-042S          | F2-3256-042S        | 42mm        | 32mm*       |
| F1-3256-044S          | F2-3256-044S        | 44mm        | 32mm*       |
| F1-3256-046S          | F2-3256-046S        | 46mm        | 32mm*       |
| F1-3256-048S          | F2-3256-048S        | 48mm        | 32mm*       |
| F1-3256-050S          | F2-3256-050S        | 50mm        | 32mm*       |
| F1-3256-055S          | F2-3256-055S        | 55mm        | 32mm*       |
| F1-3256-060S          | F2-3256-060S        | 60mm        | 32mm*       |
| F1-3256-065S          | F2-3256-065S        | 65mm        | 32mm*       |
| F1-3256-070S          | F2-3256-070S        | 70mm        | 32mm*       |
| F1-3256-075S          | F2-3256-075S        | 75mm        | 32mm*       |

**\*Note: Special Order**

# Catalog Information – Implants

## Ø7.4mm Headless Compression Screw / Headed Screw



| Headless Ø7.4mm (REF) | Headed Ø7.4mm (REF) | Length (mm) | Thread (mm) |
|-----------------------|---------------------|-------------|-------------|
| F1-1974-040S          | F2-1974-040S        | 40mm        | 19mm        |
| F1-1974-042S          | F2-1974-042S        | 42mm        | 19mm        |
| F1-1974-044S          | F2-1974-044S        | 44mm        | 19mm        |
| F1-1974-046S          | F2-1974-046S        | 46mm        | 19mm        |
| F1-1974-048S          | F2-1974-048S        | 48mm        | 19mm        |
| F1-1974-050S          | F2-1974-050S        | 50mm        | 19mm        |
| F1-1974-055S          | F2-1974-055S        | 55mm        | 19mm        |
| F1-1974-060S          | F2-1974-060S        | 60mm        | 19mm        |
| F1-1674-065S          | F2-1674-065S        | 65mm        | 19mm        |
| F1-1974-070S          | F2-1974-070S        | 70mm        | 19mm        |
| F1-1974-075S          | F2-1974-075S        | 75mm        | 19mm        |
| F1-1974-080S          | F2-1974-080S        | 80mm        | 19mm        |
| F1-1974-085S          | F2-1974-085S        | 85mm        | 19mm        |
| F1-1974-090S          | F2-1974-090S        | 90mm        | 19mm        |
| F1-1974-095S          | F2-1974-095S        | 95mm        | 19mm        |
| F1-1974-100S          | -                   | 100mm       | 19mm        |

### Screw Washer options

|   | REF          | Description              |
|---|--------------|--------------------------|
|  | F2-0025-000S | Washer Ø2.5mm / OD 5mm   |
|  | F2-0043-000S | Washer Ø4.3mm / OD 7.5mm |
|  | F2-0056-000S | Washer Ø5.6mm / OD 11mm  |
|  | F2-0074-000S | Washer Ø7.4mm / OD 13mm  |

**Ø7.4mm** Headless Compression Screw / Headed Screw



| Headless Ø7.4mm (REF) | Headed Ø7.4mm (REF) | Length (mm) | Thread (mm) |
|-----------------------|---------------------|-------------|-------------|
| -                     | F2-3274-040S        | 40mm        | 32mm*       |
| -                     | F2-3274-042S        | 42mm        | 32mm*       |
| -                     | F2-3274-044S        | 44mm        | 32mm*       |
| F1-3274-046S          | F2-3274-046S        | 46mm        | 32mm*       |
| F1-3274-048S          | F2-3274-048S        | 48mm        | 32mm*       |
| F1-3274-050S          | F2-3274-050S        | 50mm        | 32mm*       |
| F1-3274-055S          | F2-3274-055S        | 55mm        | 32mm*       |
| F1-3274-060S          | F2-3274-060S        | 60mm        | 32mm*       |
| F1-3274-065S          | F2-3274-065S        | 65mm        | 32mm*       |
| F1-3274-070S          | F2-3274-070S        | 70mm        | 32mm*       |
| F1-3274-075S          | F2-3274-075S        | 75mm        | 32mm*       |
| F1-3274-080S          | F2-3274-080S        | 80mm        | 32mm*       |
| F1-3274-085S          | F2-3274-085S        | 85mm        | 32mm*       |
| F1-3274-090S          | F2-3274-090S        | 90mm        | 32mm*       |
| F1-3274-095S          | F2-3274-095S        | 95mm        | 32mm*       |
| F1-3274-100S          | -                   | 100mm       | 32mm*       |
| F1-3274-105S          | -                   | 105mm       | 32mm*       |

**\*Note: Special Order**

# Catalog Information – Instruments








| REF | Description |
|-----|-------------|
|-----|-------------|

F4-5674-000S

CREED Instrument Kit Large for Ø5.6mm and Ø7.4mm screws

S = items are sterile



|   |   | Qty. |
|---|---|------|
|  | Countersink / Depth Gauge Large         | 1    |
|  | K-wire Ø2.8 x 230mm Drill Tip           | 2    |
|  | K-wire Ø3.2 x 230mm Drill Tip           | 2    |
|  | Universal Handle, Zimm / Hall Interface | 1    |
|  | Screwdriver Blade T25 x 140mm           | 1    |
|  | Screwdriver Blade T30 x 140mm           | 1    |
|  | Cannulated Drill Ø6.1mm x 200mm         | 1    |



| REF | Description |
|-----|-------------|
|-----|-------------|

F4-0028-230S CREED Ø2.8 K-Wire Kit

**Qty.**



K-wire Ø2.8 x 230mm Drill Tip

2

| REF | Description |
|-----|-------------|
|-----|-------------|

F4-0032-230S CREED Ø3.2 K-Wire Kit

**Qty.**



K-wire Ø3.2 x 230mm Drill Tip

2

# Creed® – Large Headless Screw System Dispenser Layout



Large Box  
Cardboard Dispenser



Large Box  
Shipping Container



## Headless Compression Screws – Large

| Part No      | Description  | Qty |
|--------------|--|-----|
| F1-1656-030S | Thread 16mm / Ø5.6mm x 30mm                        | 2   |
| F1-1656-032S | Thread 16mm / Ø5.6mm x 32mm                        | 2   |
| F1-1656-034S | Thread 16mm / Ø5.6mm x 34mm                        | 2   |
| F1-1656-036S | Thread 16mm / Ø5.6mm x 36mm                        | 2   |
| F1-1656-038S | Thread 16mm / Ø5.6mm x 38mm                        | 2   |
| F1-1656-040S | Thread 16mm / Ø5.6mm x 40mm                        | 2   |
| F1-1656-042S | Thread 16mm / Ø5.6mm x 42mm                        | 2   |
| F1-1656-044S | Thread 16mm / Ø5.6mm x 44mm                        | 2   |
| F1-1656-046S | Thread 16mm / Ø5.6mm x 46mm                        | 2   |
| F1-1656-048S | Thread 16mm / Ø5.6mm x 48mm                        | 2   |
| F1-1656-050S | Thread 16mm / Ø5.6mm x 50mm                        | 2   |
| F1-1656-055S | Thread 16mm / Ø5.6mm x 55mm                        | 2   |
| F1-1656-060S | Thread 16mm / Ø5.6mm x 60mm                        | 2   |
| F1-1656-065S | Thread 16mm / Ø5.6mm x 65mm                        | 2   |
| F1-1656-070S | Thread 16mm / Ø5.6mm x 70mm                        | 2   |
| F1-1656-075S | Thread 16mm / Ø5.6mm x 75mm                        | 2   |
| F1-1656-080S | Thread 16mm / Ø5.6mm x 80mm                        | 2   |
| F1-1974-040S | Thread 16mm / Ø7.4mm x 40mm                        | 2   |
| F1-1974-042S | Thread 16mm / Ø7.4mm x 42mm                        | 2   |
| F1-1974-044S | Thread 16mm / Ø7.4mm x 44mm                        | 2   |
| F1-1974-046S | Thread 16mm / Ø7.4mm x 46mm                        | 2   |
| F1-1974-048S | Thread 16mm / Ø7.4mm x 48mm                        | 2   |
| F1-1974-050S | Thread 16mm / Ø7.4mm x 50mm                        | 2   |
| F1-1974-055S | Thread 16mm / Ø7.4mm x 55mm                        | 2   |
| F1-1974-060S | Thread 16mm / Ø7.4mm x 60mm                        | 2   |
| F1-1674-065S | Thread 16mm / Ø7.4mm x 65mm                        | 2   |
| F1-1974-070S | Thread 16mm / Ø7.4mm x 70mm                        | 2   |
| F1-1974-075S | Thread 16mm / Ø7.4mm x 75mm                        | 2   |
| F1-1974-080S | Thread 16mm / Ø7.4mm x 80mm                        | 2   |
| F1-1974-085S | Thread 16mm / Ø7.4mm x 85mm                        | 2   |
| F1-1974-090S | Thread 16mm / Ø7.4mm x 90mm                        | 2   |
| F1-1974-095S | Thread 16mm / Ø7.4mm x 95mm                        | 2   |
| F1-1974-100S | Thread 16mm / Ø7.4mm x 100mm                       | 2   |
| F4-5674-000S | Instrument Kit Large for Ø5.6mm and Ø7.4mm screws* | 4   |
| 80-143       | Large Box Shipping Container                       | 1   |
| 40-226F      | Large Box Inlay                                    | 1   |
| 40-224F      | Large Box Cardboard Dispenser                      | 1   |
| F4-0028-230S | Ø2.8 K-Wire Kit (2 K-Wires per Kit)                | 5   |
| F4-0032-230S | Ø3.2 K-Wire Kit (2 K-Wires per Kit)                | 5   |



# Creed® – Large Headed Screw System Dispenser Layout



Large Box  
Cardboard Dispenser



Large Box  
Shipping Container



## Headed Compression Screws – Large

| Part No      | Description                                       | Qty |
|--------------|---|-----|
| F2-1656-030S | Thread 16mm / Ø5.6mm x 30mm                       | 2   |
| F2-1656-032S | Thread 16mm / Ø5.6mm x 32mm                       | 2   |
| F2-1656-034S | Thread 16mm / Ø5.6mm x 34mm                       | 2   |
| F2-1656-036S | Thread 16mm / Ø5.6mm x 36mm                       | 2   |
| F2-1656-038S | Thread 16mm / Ø5.6mm x 38mm                       | 2   |
| F2-1656-040S | Thread 16mm / Ø5.6mm x 40mm                       | 2   |
| F2-1656-042S | Thread 16mm / Ø5.6mm x 42mm                       | 2   |
| F2-1656-044S | Thread 16mm / Ø5.6mm x 44mm                       | 2   |
| F2-1656-046S | Thread 16mm / Ø5.6mm x 46mm                       | 2   |
| F2-1656-048S | Thread 16mm / Ø5.6mm x 48mm                       | 2   |
| F2-1656-050S | Thread 16mm / Ø5.6mm x 50mm                       | 2   |
| F2-1656-055S | Thread 16mm / Ø5.6mm x 55mm                       | 2   |
| F2-1656-060S | Thread 16mm / Ø5.6mm x 60mm                       | 2   |
| F2-1656-065S | Thread 16mm / Ø5.6mm x 65mm                       | 2   |
| F2-1656-070S | Thread 16mm / Ø5.6mm x 70mm                       | 2   |
| F2-1656-075S | Thread 16mm / Ø5.6mm x 75mm                       | 2   |
| F2-1974-040S | Thread 16mm / Ø7.4mm x 40mm                       | 2   |
| F2-1974-042S | Thread 16mm / Ø7.4mm x 42mm                       | 2   |
| F2-1974-044S | Thread 16mm / Ø7.4mm x 44mm                       | 2   |
| F2-1974-046S | Thread 16mm / Ø7.4mm x 46mm                       | 2   |
| F2-1974-048S | Thread 16mm / Ø7.4mm x 48mm                       | 2   |
| F2-1974-050S | Thread 16mm / Ø7.4mm x 50mm                       | 2   |
| F2-1974-055S | Thread 16mm / Ø7.4mm x 55mm                       | 2   |
| F2-1974-060S | Thread 16mm / Ø7.4mm x 60mm                       | 2   |
| F2-1974-065S | Thread 16mm / Ø7.4mm x 65mm                       | 2   |
| F2-1974-070S | Thread 16mm / Ø7.4mm x 70mm                       | 2   |
| F2-1974-075S | Thread 16mm / Ø7.4mm x 75mm                       | 2   |
| F2-1974-080S | Thread 16mm / Ø7.4mm x 80mm                       | 2   |
| F2-1974-085S | Thread 16mm / Ø7.4mm x 85mm                       | 2   |
| F2-1974-090S | Thread 16mm / Ø7.4mm x 90mm                       | 2   |
| F2-1974-095S | Thread 16mm / Ø7.4mm x 95mm                       | 2   |
| F2-0056-000S | Washer Ø5.6mm / OD 11mm                           | 4   |
| F2-0074-000S | Washer Ø7.4mm / OD 13mm                           | 4   |
| F4-5674-000S | Instrument Kit Large for Ø5.6mm and Ø7.4mm screws | 4   |
| 80-143       | Large Box Shipping Container                      | 1   |
| 40-226F      | Large Box Inlay                                   | 1   |
| 40-224F      | Large Box Cardboard Dispenser                     | 1   |
| F4-0028-230S | Ø2.8 K-Wire Kit (2 K-Wires per Kit)               | 5   |
| F4-0032-230S | Ø3.2 K-Wire Kit (2 K-Wires per Kit)               | 5   |



# Notes

# Notes



CAUTION: Federal (USA) law restricts this device to sale by or on the order of a surgeon. Rx only.

This document is intended solely for the use of healthcare professionals. This technique was developed in conjunction with healthcare professionals. A surgeon must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. GLW, Inc. does not dispense medical advice and recommends that surgeons be trained in the use of any particular product before using it in surgery. The information presented is intended to demonstrate a GLW, Inc. product. A surgeon must always refer to the package insert, product label and/or instructions for use, including the instructions for Cleaning and Sterilization (if applicable), before using any GLW, Inc. product.

GLW, Inc., GLW Medical Innovation, Carbon22 and Creed are trademarks of GLW, Inc.. Innov8ortho is the exclusive distributor of the Creed Ortholuent Implants.

**[www.glwmed.com](http://www.glwmed.com)**

© 2023 GLW, Inc. All rights reserved.

Patents: [www.glwmed.com/patents/](http://www.glwmed.com/patents/)



Scan for more  
product information



**Legal Manufacturer:**

GLW, Inc.  
300 Sylvan Ave, 2<sup>nd</sup> Floor  
Englewood Cliffs, NJ 07632



**Distributed by:**

Innov8ortho, LLC  
300 Sylvan Ave, 2nd Floor  
Englewood Cliffs, NJ 07632  
[custsvc@innov8ortho.com](mailto:custsvc@innov8ortho.com)

Ref: CRERF-OT-Ed2-06-23-EN