

hammerthread™

Lesser toe deformity correction



Seeing is **Believing**


GLW Foot & Ankle



Effective and reliable

HammerThread is a $\text{\O}2.5\text{mm}$ cannulated, titanium implant, designed specifically for the correction of smaller toe deformities.

A small and effective portfolio of implants with appropriate design and length options for proximal and distal interphalangeal joint (PIPJ-DIPJ) arthrodesis.

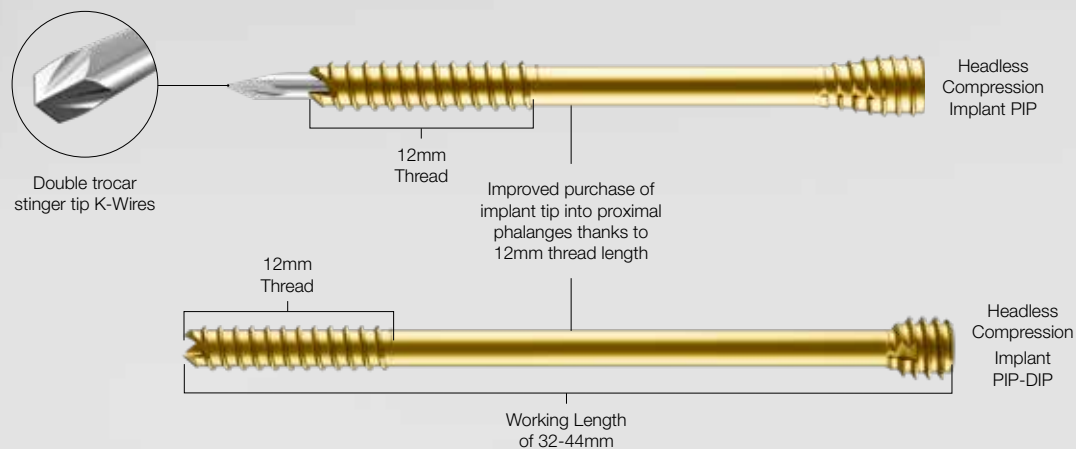


The length span of 32–44mm allows the appropriate fixation of PIP and DIP joints for a wide range of anatomies.

Superior precision

The HammerThread implants were designed to provide surgeons with a minimally invasive and reliable solution to address smaller toe deformities.

The system is comprised of $\text{\O}2.5\text{mm}$ titanium Ti64 implants that accept $\text{\O}1.3$ K-Wires. This combination results in superior precision during insertion and ample stability during the reduction of proximal and distal interphalangeal joints (PIPJ-DIPJ).

The double stinger tip design of the K-Wires also allows for retrograde insertion of the HammerThread implants for both PIPJ and DIPJ fixation.

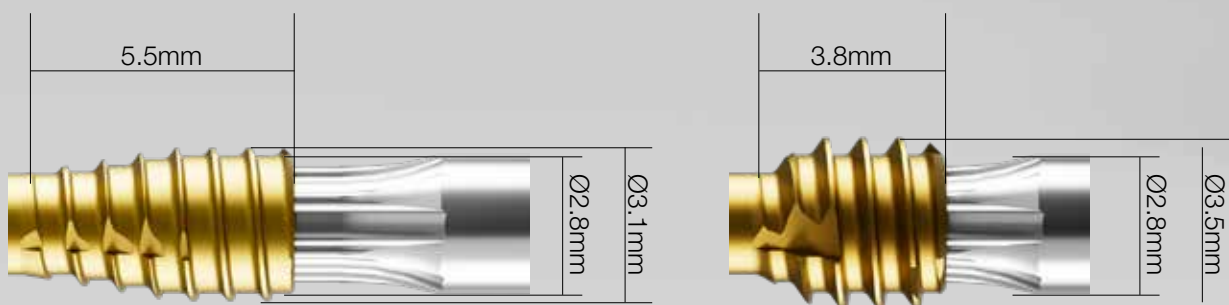

Reverse-Cutting Design
Expedites implant extraction



Forward-Cutting Design
Accelerates screwhead countersinking




Large Cannulation
Accepts Ø1.3mm K-Wires



Tapered Head Design of PIP Screws
Seamless fixation of PIPJ



Torx Interface
Promotes a sturdy driver /implant interface

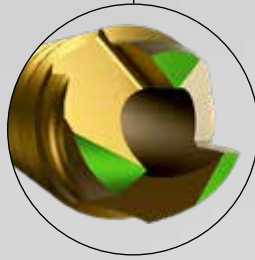

Self-Drilling Tip Design

Aggressive Tri-tip design for unmatched cutting performance



Self-Tapping Tip Design




Reverse-Cutting Feature
Facilitates removal of implant



Ready-to-use, everytime.

The instrumentation is provided in a sterile, ready-to-use instrument kit designed specifically for the correction of smaller toe deformities.

The instrument kit includes 4x double stinger tip $\text{\O}1.3\text{mm}$ K-Wires, a mini depth gauge, an AO-handle and a cannulated T8 driver with a tapered shaft that matches the implants outer diameter and allows the compression of the PIPJ only without requiring additional hole preparation.

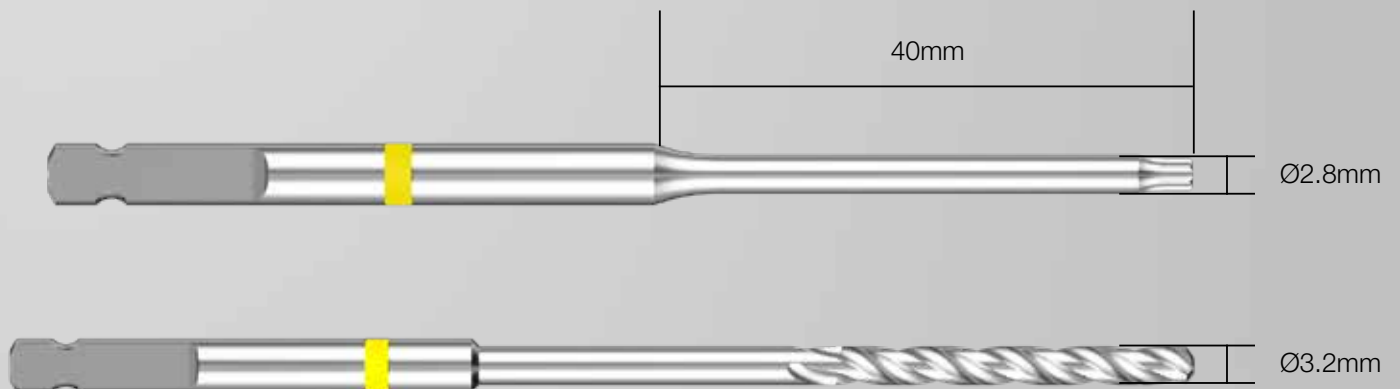

hammerthreadTM

Lesser toe deformity correction



Scan for more
product information

CAUTION: Federal (USA) law restricts this device to sale by or on the order of a surgeon. Rx only.

GLW, Inc, GLW Medical Innovation, Carbon22 and Hammerthread are trademarks of GLW, Inc. Innov8ortho is the exclusive distributor of the Hammerthread Lesser Toe Deformity Correction.

www.glwmed.com

© 2023 GLW, Inc. All rights reserved.
Patents: www.glwmed.com/patents/



Legal Manufacturer:

GLW, Inc.
300 Sylvan Ave, 2nd Floor
Englewood Cliffs, NJ 07632

innov8ortho[®]

Distributed by:

Innov8ortho, LLC
300 Sylvan Ave, 2nd Floor
Englewood Cliffs, NJ 07632
custsvc@innov8ortho.com

Ref: HAM-OT-Ed1-04-23-EN